

ONETOUCH Reveal®



OneTouch Reveal® web app


Helps you get more meaning out of your results

- Tracks your blood glucose, food, insulin dosing and activity data over time
- Suggests personalized tips to help you stay on track and in range*
- Reports can be printed, ready for discussion at your next appointment with your healthcare professional
- Can be used on PC and Mac computers at OneTouchReveal.com

* Be sure to talk to your healthcare professional about the actions that are right for you.

How to setup your OneTouch Reveal® web app

Before you begin, confirm that cookies are enabled in your browser.

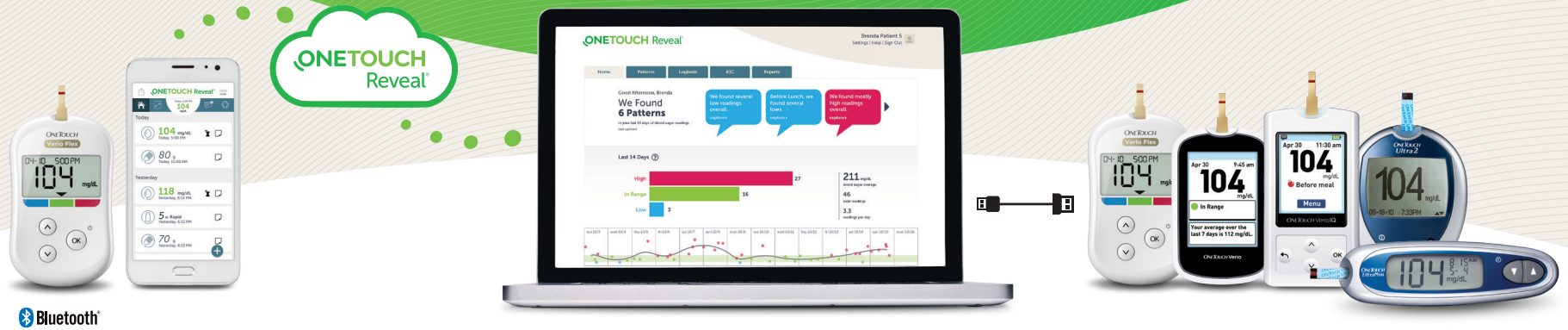

Wireless Option

1. Connect wireless devices and activate your account

- Install the OneTouch Reveal® mobile app on your mobile device
- Open the mobile app and follow the on-screen instructions to setup your account and pair your OneTouch Verio Flex® meter
- Check your email inbox for a link to confirm your email address

For more details, refer to the OneTouch Reveal® mobile app **Help File**.

2. Sign in to the OneTouch Reveal® web app on your computer

- Go to **OneTouchReveal.com** and enter the same **Username** and **Password** that you used in the OneTouch Reveal® mobile app
- Click **Sign In** and follow the on-screen instructions

OR

Cable Option

1. Register and activate your account

- Go to **OneTouchReveal.com**, click on **Create a New Account** and follow the prompts
- Check your email inbox for a link to confirm your email address

2. Install Data Transfer Tool


- Unplug all OneTouch® USB cables from your computer
- **Sign In** to your OneTouch Reveal® account and click **help** from the top right of any screen
- Click the **Download** button under **Data Transfer Tool** and follow the prompts
- Select the appropriate devices
- Click **Finish**. After installation, you may be prompted to re-start your computer.

3. Transfer data from your OneTouch® meter

- Connect your meter to the computer with your OneTouch® USB cable (*cable sold separately*).
- Open the OneTouch Reveal® Data Transfer Tool and select **Detect Now**. When the device is found, a confirmation screen will appear.
- **Sign In** to the OneTouch Reveal® web application
- Click **Confirm** to transfer data. When the transfer completes, click **OK**




You're all set! You can now access your data with the OneTouch Reveal® web app at **OneTouchReveal.com**

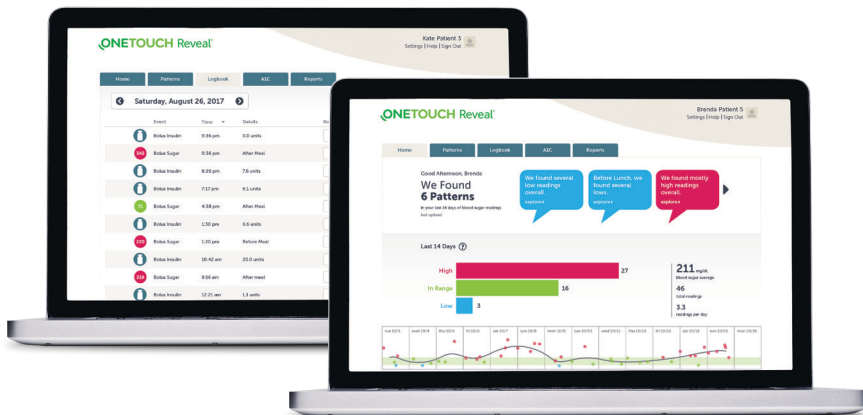

View your blood glucose readings

- Click the **Logbook** tab to see your blood glucose readings
- Click the **Patterns** tab to view a 14-day summary

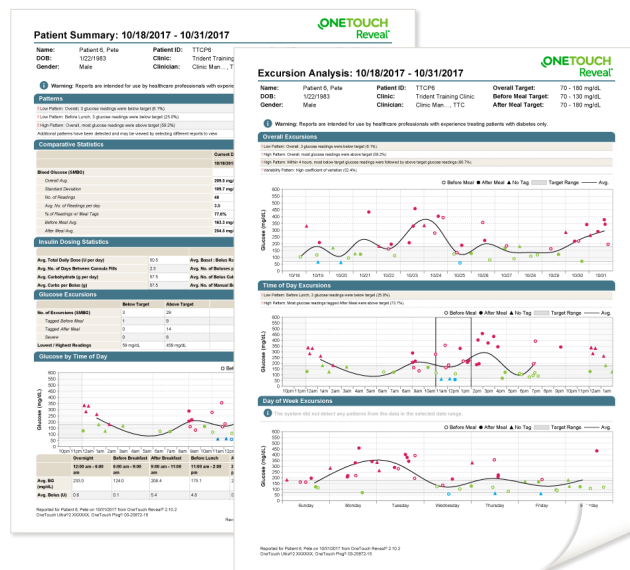

Prepare for your next visit with your healthcare professional

- Click **Visiting your doctor?**
The **Be prepared for your visit** screen will appear.
- Click **Download reports**
To include reports, be sure to click the box next to **Include reports for your doctor** before you click **Download reports**.

If you'd like to change the list of reports you want to print for your visit, you can do so by clicking on **Settings**, then **Preferred Reports to Print** and then selecting the reports you need.

Any questions? We're here to help



Visit **OneTouch.com**



Call us at 1 866 693-0599

Find the necessary system requirements at
OneTouchReveal.com/support/en_US/



ONETOUCH[®]
every touch is a step forward[®]