


Let's get started with the OneTouch Reveal® web app

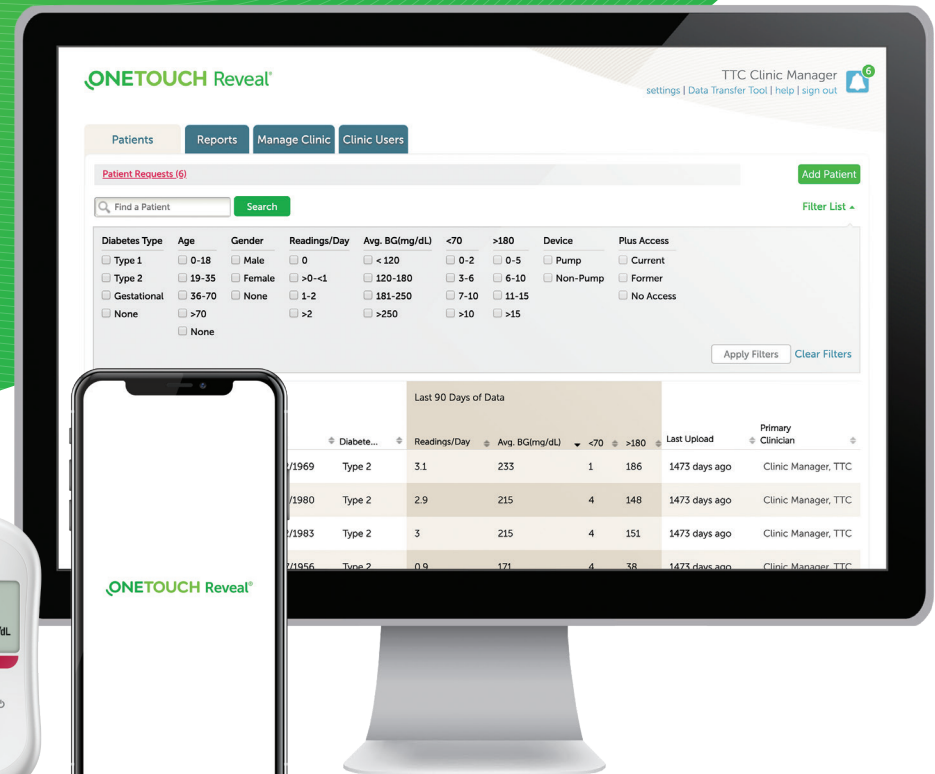

Step-by-Step Guide

Your step-by-step guide to setting up and using the OneTouch Reveal® web app with the OneTouch Reveal® mobile app




For Healthcare Professionals only

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by LifeScan Scotland Ltd. and its affiliates is under license. Other trademarks and trade names are those of their respective owners. © 2020 LifeScan IP Holdings, LLC US-WDM-2000010

ONETOUCH
every touch is a step forward®

The OneTouch Reveal® web app – supporting you and your patients in tracking data and managing diabetes

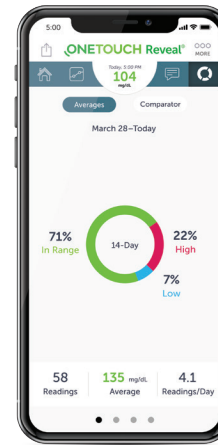

OneTouch Reveal® mobile app

- Automatically synchronizes data from your OneTouch Verio Reflect® or OneTouch Verio Flex® meter helping you **spend less time looking for blood glucose patterns and trends.**
- **Connects blood glucose, food, insulin and activity,** data in one convenient place, accessible when you sync.



Designed to help you see what's working and what's not for your patients

- Helps track glucose and insulin data, and detect patterns.
- Creates dynamic, colorful reports based on preferences set and data available such as target ranges and devices connected when reports are run.
- Seamlessly gathers data from all the devices that are linked to a patient's account.

Helping your patients to stay on track between visits

- Glucose monitoring and insulin dosing pattern messages detected from patient devices appear in reports and can be used as a guide for managing diabetes.
- Reports can be printed to be ready for discussion at their next visit.

For Healthcare Professionals only

Apple and the Apple logo are trademarks of Apple Inc., registered in the US and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. Other trademarks and trade names are those of their respective owners. © 2020 LifeScan IP Holdings, LLC US-WDM-2000010


ONETOUCH®
every touch is a step forward®

Quick steps to setting up and using the OneTouch Reveal® web app

It assists you and your patients

1 Sign in (must have account activated)


- Open **OneTouchReveal.com** using your browser
- Enter your Username and Password
- Verify that the current language is correct
- Click **Sign In**

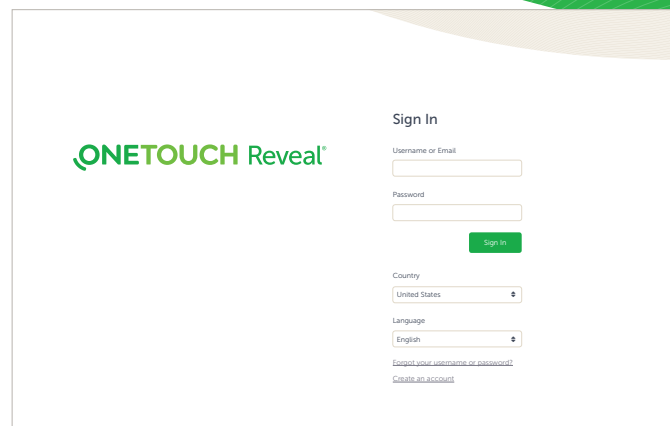

2 Add new patients

Option 1 – Clinic creates patient account

- Click the **Patients** tab, then click **Add Patient**
- Print **Patient Consent Form** from the link in the HCP Disclaimer consent for the patient to sign
- Complete **HCP Disclaimer consent**
- Fill in the **Add a New Patient** fields (* indicates mandatory field)
- Click **Save**
- The **Registration Summary** page will load, confirming that the account has been created and that the patient has been added to the clinic
- Click **Print Registration Summary** to provide your patients with instructions on how to activate their accounts

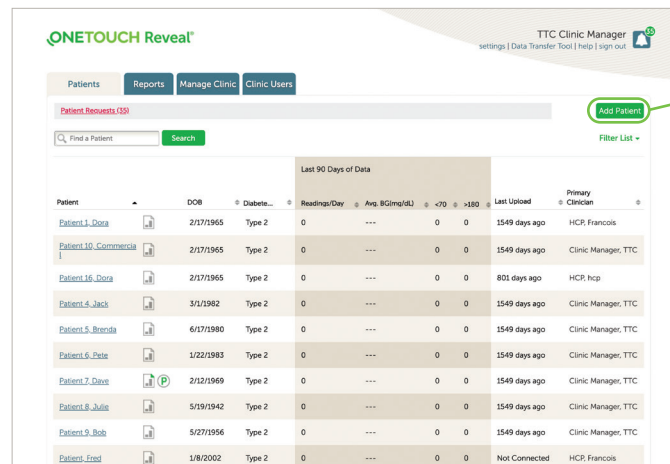

Patient	DOB	Diabetes	Readings/Day	Avg. %Clim(dL)	Last Upload	Primary Clinician
Patient 1 Data	2/27/1965	Type 2	0	---	0 0	1549 days ago HCP Francois
Patient 10 Commercial	2/27/1965	Type 2	0	---	0 0	1549 days ago Clinic Manager, TTC
Patient 16 Data	2/27/1965	Type 2	0	---	0 0	801 days ago HCP hcp
Patient 4 Jack	3/2/1982	Type 2	0	---	0 0	1549 days ago Clinic Manager, TTC
Patient 5 Brenda	6/27/1980	Type 2	0	---	0 0	1549 days ago Clinic Manager, TTC
Patient 6 Pete	1/22/1983	Type 2	0	---	0 0	1549 days ago Clinic Manager, TTC
Patient 7 Dave	2/27/1969	Type 2	0	---	0 0	1549 days ago Clinic Manager, TTC
Patient 8 Julie	5/19/1942	Type 2	0	---	0 0	1549 days ago Clinic Manager, TTC
Patient 9 Bob	5/27/1956	Type 2	0	---	0 0	1549 days ago Clinic Manager, TTC
Patient Fred	1/8/2002	Type 2	0	---	0 0	Not Connected HCP Francois

Add Patient

Quick steps to setting up and using the OneTouch Reveal® web app

It assists you and your patients

2 Add new patients

Option 2 – Link a patient to your clinic

- Instruct patient to install the OneTouch Reveal® mobile app and follow on-screen prompts to create an account
- Once installed, ensure patient pairs their OneTouch Verio Reflect® or OneTouch Verio Flex® meter with the mobile app
- Provide patient with your specific 6 digit Clinic Account Office Code and ask them to send a request to share data with your clinic
- Click on the Notification icon on your Profile picture, to access pending actions. Under the Patient Requests section, you will see a list of pending requests
- Click **Confirm** to accept request. Add **Patient ID** to help identify this patient in your records

For support, patients can refer to the meter Owner's Booklet or call 1-866-693-0599

Need help finding your Clinic Account Code? Click on **Manage Clinic** tab. Then go under **Settings > My Clinic > Clinic Code**



Enter Clinic Code:

--	--	--	--	--	--



Encourage your patients to activate their accounts so they can share their data

For Healthcare Professionals only

Apple and the Apple logo are trademarks of Apple Inc., registered in the US and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by LifeScan Scotland Ltd. and its affiliates is under license. Other trademarks and trade names are those of their respective owners.

© 2020 LifeScan IP Holdings, LLC US-WDM-2000010


ONETOUCH®
every touch is a step forward™

Quick steps to setting up and using the OneTouch Reveal® web app

It assists you and your patients


3 View reports

- Click the **Reports** tab in the OneTouch Reveal® web app
- Enter any part of the patient's name or ID in the **Search for Patient** bar, or sort by clicking on the top of any column
- Click on the patient's name
- You can now choose and view patient reports

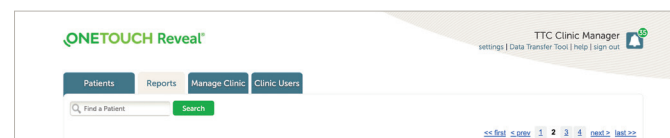

4 Download and print reports


- Click **Download Reports** from the report you are viewing
- Choose a report appropriate for your consultation
- A PDF of the report will download. Double-click on the PDF to open and then print

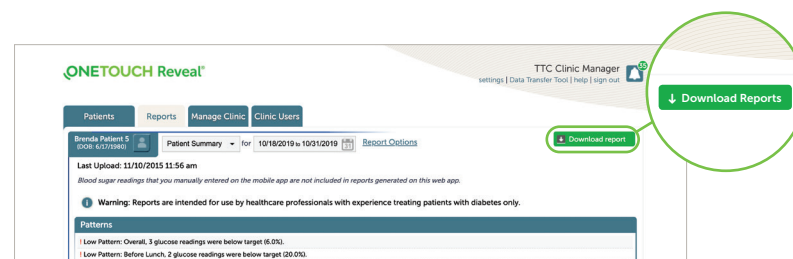

5 Remotely monitor patient progress

The Patient Population Management Tool is designed so that you can filter and sort your patient information in meaningful ways:

- Patients with Type 1 diabetes with the most lows (< 70 mg/dL)
- Patients with Type 2 diabetes with the most highs (> 180 mg/dL)
- Average blood glucose readings for Type 1 and Type 2 patient populations
- Further refine your list by sorting for number of blood glucose readings per day
- Age, gender, etc.

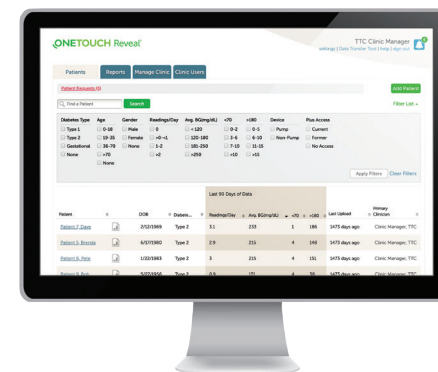

ONETOUCH
every touch is a step forward™

Any questions? We're here to help



Visit **OneTouch.com**



Call us at 1-866-693-0599

Find the necessary system requirements at
OneTouchReveal.com/support/en_US/

