

The OneTouch® Delica® Lancing System.[†] Designed for Comfortable Testing.

Lancets available in two sizes (30G Fine and 33G Extra Fine).



1 Remove cap (**twist counterclockwise** and **pull**) and insert lancet until it snaps into place.



2 Twist off the plastic lancet top with **one full turn**. Replace the cap; **turn clockwise** to secure.



3 **Set depth control:** 1-7 (1 is the shallowest setting. 7 is the deepest).



4 **Slide the cocking control back until it clicks.** Hold device firmly against the side of your finger. Press the round release button.

Important: Before lancing, read the OneTouch® Delica® User Guide carefully. Lancets are sterile and for single use only to reduce the risk of infection.

Q: How can you make testing simple?

A: With the OneTouch Verio® Meter.



The OneTouch Verio® Meter helps you better understand your results, without any extra work.

Questions?

Call 1-888-567-3003 or visit www.OneTouch.com

[†]Uses only OneTouch® Delica® Lancets.
All trademarks are used under license by LifeScan, Inc.
© 2014 LifeScan, Inc., 965 Chesterbrook Blvd, Wayne, PA 19087 LFS-14-4350A 10/14

ONETOUCH®

Get Results with the OneTouch Verio® Meter

In 2 Easy Steps:

Before testing, wash hands and dry completely.

1 Insert a test strip

Insert a test strip into the test strip port with the gold side of the test strip and the two silver prongs facing you.

No coding is required.



2 Apply blood

When the 'Apply Blood' screen appears on the display, you can apply your blood sample to either side of the test strip.



Get Result

Your blood glucose result appears on the display along with a color dot indicating if you are in, above or below range.



IMPORTANT: These instructions are simplified to supplement training by your healthcare professional. Before testing, read your OneTouch® Meter Owner's Booklet carefully.